

PAT 10™ Medical Drying Cabinet

Allied's PAT 10 Medical Drying Cabinet circulates clean dry air at high temperatures of 115°F (46°C) to 150°F (66°C), making it capable of drying a full load of medical hoses, tubing, masks and other equipment in as little as 35 minutes. The design features louvered walls that create a unique air flow pattern that efficiently dries the contents, while the powerful air flow generator heats and circulates air at a rate of at least four air exchanges per minute.

The unit features standard timed drying, as well as a unique thermostatic timed drying mode designed for equipment like hoses that trap moisture and require intense drying. The thermostatic timed drying mode ensures that the parts are dried at a temperature of above 115°F (46°C) to vaporize and remove all trapped moisture.

Operation of the PAT 10 is simple. Simply load the drying cabinet, select the drying mode, and set the timer to activate the cycle. When the timer expires, the PAT 10 will automatically shut itself off.

The PAT 10 features corrosion-resistant steel and aluminum construction and has two clear polycarbonate doors for front or rear loading. The cabinet contains 12 cubic feet of drying space and comes with 15 removable drying trays, enabling simultaneous drying of hoses and equipment.



PAT 10™ Medical Drying Cabinet

12-hour Electronic
Timer



Drying Mode
Selector

SPECIFICATIONS

Dimensions:	20" W x 20" D x 85.8" H (50.8 x 50.8 x 204.8 cm)
Drying Area:	12 cubic feet (20,736 cubic inches) 0.3 cubic meters (339,588.5 cubic centimeters)
Drying Trays:	Half-size: 16.5" W x 8.25" D x 2" H (41.9 x 41.9 x 5.08 cm) Full-size: 16.5" W x 16.5" D x 2" H (41.9 x 21.2 x 5.08 cm)
Voltages:	Model T6700: 115V/60 Hz Model T6769: 220V/50 Hz
Power Dissipation:	1380W

ORDERING INFORMATION

T6700	PAT 10 Medical Drying Cabinet, with casters, 5 full-size trays, 10 half-size trays, electric timer, automatic heat control, dryer/aerator switch circuit breaker; 115V/60 Hz
T6769	PAT 10 Medical Drying Cabinet, same as T6700; 220V/50 Hz

ACCESSORIES

T2001	Drying Tray, full size
T1844.1	Air Filter, 6" diameter, 12/case
T1844.2	Air Filter, 6" diameter, 100/case



Half-size and full-size
Drying Trays



Allied Healthcare Products, Inc. ▪ 1720 Sublette Avenue ▪ St. Louis, MO 63110 USA
314/771-2400 ▪ 800/444-3954 ▪ FAX: 314/771-0650 ▪ www.alliedhpi.com

U.S.
Toll Free: 800/444-3940
FAX: 800/477-7701

Canada & Mexico
Toll Free: 800/446-0552
FAX: 800/246-6201

International
Telephone: 314/268-1683
FAX: 314/771-5183

ISO 13485: 2003

©Allied Healthcare Products, Inc.®

All specifications are nominal and
subject to change without notice.

Form 10-00-0082
August 2013